

5


Maintenance and Care

Maintenance and care of your device is important to sustaining excellent performance and long life of the device.

Clean the surface of the device with the brush provided. Please use the tools provided in the package only.

The domes can be cleaned with dish soap and water or replacement domes can be ordered.

The thin tube can be cleaned by inserting the thin tube cleaning wire all the way through the tube.

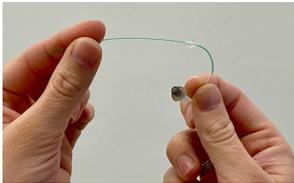

6


Replacing and Installing Domes

A dome may be removed to be replaced or to change sizes. To remove the dome:


1. Hold the thin tube in one hand and pinch the end or side of the dome with the other hand.
2. Pull the dome straight out from the thin tube.

To install a new dome:

1. Hold onto the end of the dome and push the dome over the barbs of the thin tube.
2. Ensure the dome is **completely** seated by gently pulling on it to ensure it is secure.

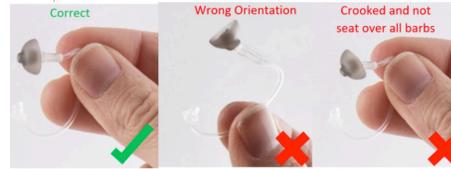

Device Overview

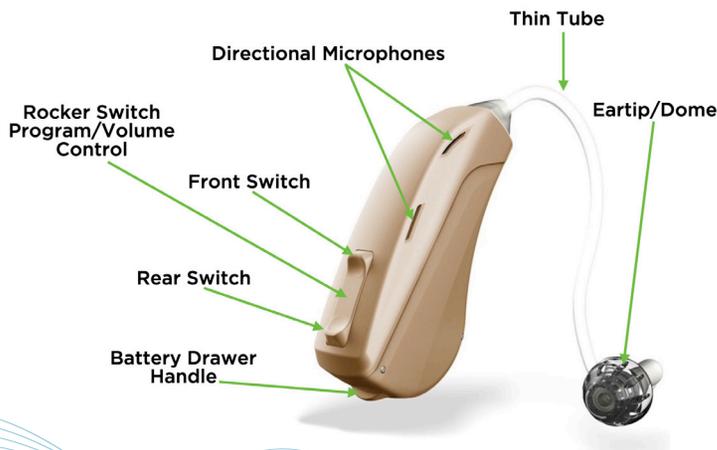

Adjusting to your new hearing

The most important thing for a successful re-entry into the world of better hearing is consistent use of your new products.

At first, you will probably be surprised how crisp your hearing sounds. Adjust the volume control to find the comfortable level by having someone speak with you in a relatively quiet environment.

If you find yourself getting fatigued in your first few days, it is okay to take a break. Wear your instruments as long as you can until you are able to use them all day.



CONTACT US!



Online: www.discounthearing.com



eMail: info@discounthearing.com



Give us a call: 1-888-901-6411

1

Turning your devices on and off

1. Use the battery drawer handle to open the drawer to the fully opened position.



2. Take a new #312 battery, pull off the brown tab and place the battery into the battery drawer with the “+” sign facing up.



Remove brown tab



3. Push the battery drawer to the on position. The devices will power on then start amplifying sound after the 5 second delay.



When the device is not in use, use the battery drawer handle to open the battery drawer to the OFF position as shown below.




When the device battery is low, you will hear a voice indicator. This will let you know its time to change your battery.

2

Placing the devices on your ears

Identify the right and left hearing devices as shown by the blue L and the red R on the tubes

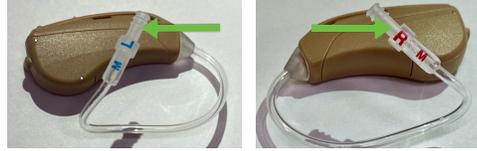

1. Place the hearing aid behind your ear and hold it in place.

2. Using the other hand if needed, gently insert the dome into your ear canal until the thin tube rests against your head.

3. Ensure the dome is seated securely in your ear and the device is resting comfortably behind your ear.

If the dome is loose in your ear canal, change to a larger size dome included in the package with your devices. See the user manual for details.





3

Adjusting the Volume

Your hearing aids always power on to the mid-point volume setting (6). Press and release the front switch to increase and the rear switch to decrease the volume.


A tone indicator will sound when the volume changes. A voice indicator will alert you that the minimum or maximum volume has been reached.

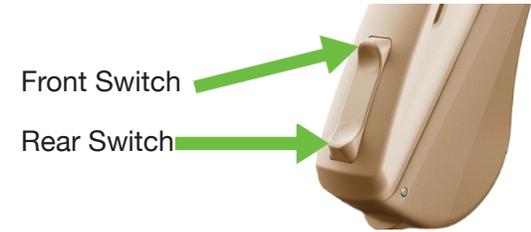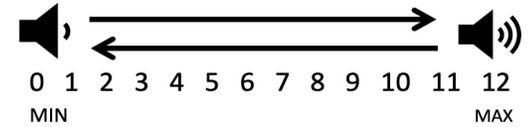

4

Changing Programs

Programs are settings inside your devices that allow for optimal performance in different environments. There are four programs in each device:

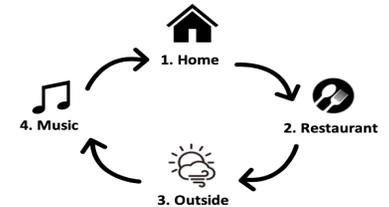

To change the program, press and hold the front or rear switch for one second and release. The voice indicators will let you know which program you have selected.